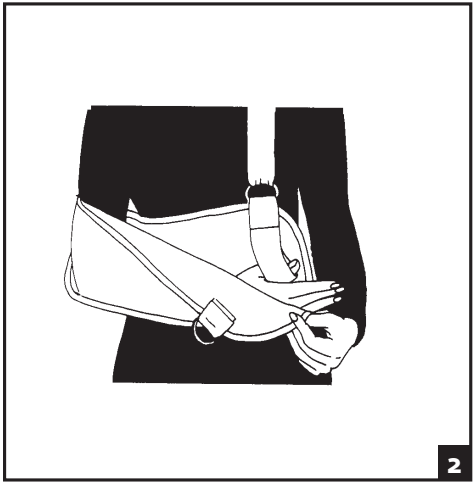
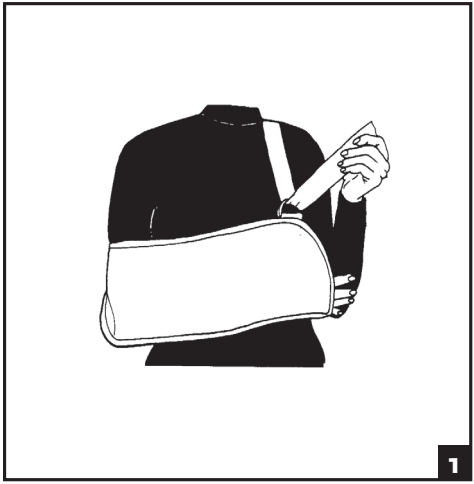
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: The Procure Deluxe Arm Sling with Pad is designed to support and immobilize the arm, shoulder and elbow joints.

PERFORMANCE CHARACTERISTICS: Soft-goods/ semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS:

- We recommend that the first application be made under the supervision of a healthcare professional.
- The support should only be used after consulting a health care professional if you suffer from skin diseases, injuries, lymphatic disorders, indistinct soft tissue swelling, or circulatory or sensory disorders in the affected area.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should stop using the device and contact your healthcare professional immediately.
- Do not use over open wounds.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues
- Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- NOTE:** Universal Arm Sling envelope length may be shortened if necessary by folding the open end inside itself and securing the hook and loop closures. Apply as noted below.
- Position elbow in the pocket of the sling. The wrist and 3/4 of the hand should be supported within the pocket.
 - If your sling pocket has a thumb loop, you may place thumb in loop to help maintain position of forearm.
 - Place shoulder straps over opposite shoulder, feed through the D-ring and secure. Hand should be slightly elevated above elbow.

CLEANING INSTRUCTIONS:

Hand wash liner in warm water using mild soap, rinse thoroughly. **AIR DRY. NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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