




HemoCue® Hb 201+ System



Confidence in your answers at the point of care

The HemoCue® Hb 201+ System brings your practice lab-quality hemoglobin results in just three simple steps: fill the microcuvette with capillary, venous or arterial whole blood sample; place it into the analyzer; and view results in about a minute. With dedicated support and service based on more than 35 years of experience, you can count on HemoCue to provide rapid results so you can make decisions when you need them most: at the point of care.

HemoCue OnCue™ Support

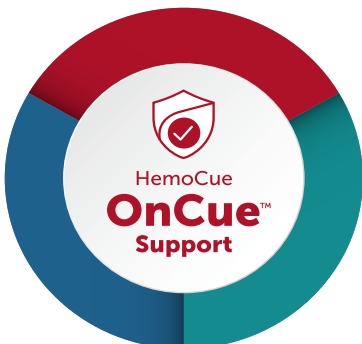


HemoCue OnCue™ Support is a comprehensive program designed to meet your needs as quickly and conveniently as possible.

Powered by HemoCue employees, you get answers and education from the experts who know our products best.

At your office, online, or on the phone, we're ready to do more for you.

HemoCue® Hb 201+ System Specifications¹

Principle	Modified azidemethemoglobin reaction; dual wavelengths (570 nm and 880 nm) for compensation of turbidity
Calibration	Factory calibrated against the ICSH reference method; needs no further calibration
Sample Material	Capillary, venous or arterial whole blood
Measurement Range	0–25.6 g/dL (0–256 g/L, 0–15.9 mmol/L)
Results	Within 60 seconds
Sample Volume	~10 µL
Dimensions	3.35 x 6.30 x 1.69 inches (85 x 160 x 43 mm)
Weight	0.77 pounds (350 g) with batteries installed
Storage Temp.	Analyzer: 32–122 °F (0–50 °C) Microcuvettes: unopened 59–86 °F (15–30 °C); three-month open vial stability
Operating Temp.	59–86 °F (15–30 °C)
Power	AC Adapter or 4 AA batteries
Interface	Printer and HemoCue® Basic Connect including optional barcode scanner
Quality Control	Built-in self-test; optional liquid controls*
CPT Code	85018-QW <i>The CPT codes provided are based on AMA guidelines and are for informational purposes only. CPT coding is the sole responsibility of the billing party. Please direct any questions regarding coding to the payer being billed.</i>

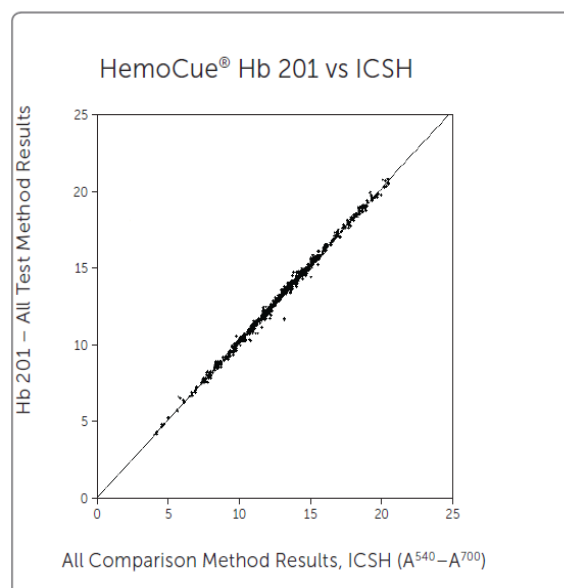


Method comparison according to CLSI EP9-A.
HemoCue® Hb 201+ System vs ICSH, venous blood samples, g/dL.²


Product Order Number

121721	HemoCue® Hb 201+ Analyzer
111716	HemoCue® Hb 201 Microcuvettes (200/box)
111715	HemoCue® Hb 201 Individually Packaged Microcuvettes (100/box)

Get Results in a Few Simple Steps

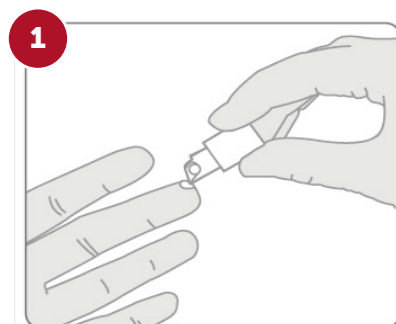

Fill the microcuvette.

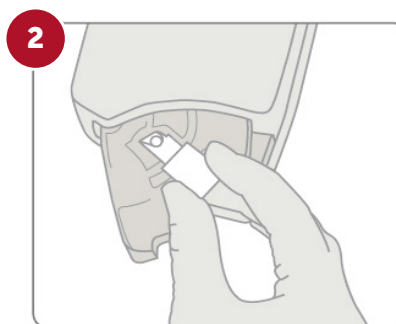

Place the microcuvette into the analyzer.



View results.

*Refer to your local or regulatory agency for any external control requirements

¹ HemoCue® Hb 201+ System Operating Manual 901705 180917

² Method Comparison and Bias Estimation using patient samples according to NCCLS EP-9 for Hb 201+ versus ICSH ($A^{540}-A^{700}$), Site A-D, Multicenter Study (FRE0807/1).

HemoCue America | California, USA | Phone 800.881.1611 | Fax 800.333.7043

web@hemocue.com hemocue.us

©2020 HemoCue America. All rights reserved. MMUS-0116[4]

