

**SAM**  
MEDICAL

# SAM<sup>®</sup> Pelvic Sling II

**SLIDE.  
CLICK.  
SECURE.**

## OPTIMAL FORCE STABILIZATION

SAM<sup>®</sup> Pelvic Sling is the first and only force-controlled circumferential pelvic belt. Featuring an optimized one-piece design, it's scientifically proven to safely and effectively reduce and stabilize open-book pelvic ring fractures.



## FEATURED TECHNOLOGY

### AUTOSTOP BUCKLE

The patented SAM<sup>®</sup> AUTOSTOP buckle acts to avoid over- or under-tightening of SAM<sup>®</sup> Pelvic Sling. Once optimal compressive force is reached, two prongs activate, clicking to confirm correct application. The sling is secured in place with its hook and loop system strap.






# SAM<sup>®</sup> Pelvic Sling II

## ENGINEERED TO PRESERVE LIFE

- 1 LOW-FRICTION POSTERIOR SLIDER**  
Easier transfers following severe trauma made possible by low-friction materials covering backside of the pelvic sling.
- 2 SAM<sup>®</sup> AUTOSTOP BUCKLE**  
Tactile and audible “click” provides clear feedback, confirming proper application and preventing over-tightening while maintaining optimal force.
- 3 PRECISION STRAP**  
Precisely-engineered holes lock with AUTOSTOP Buckle, allowing for quick application. Secure fastening with sturdy hook and loop strap.

- 4 TAPERED DESIGN**  
Narrow and tapered front facilitates urinary catheterization, interventional radiology, external fixation, and abdominal surgery without removal.
- 5 RADIOLUCENT MATERIAL**  
All materials are radiolucent except the buckle’s two stainless steel springs, allowing X-rays and CT-scans without removal.
- 6 3 SIZES AVAILABLE**  
The Standard size fits 95% of the population. Complete the set with the Small and Large sizes to fit almost any body type.

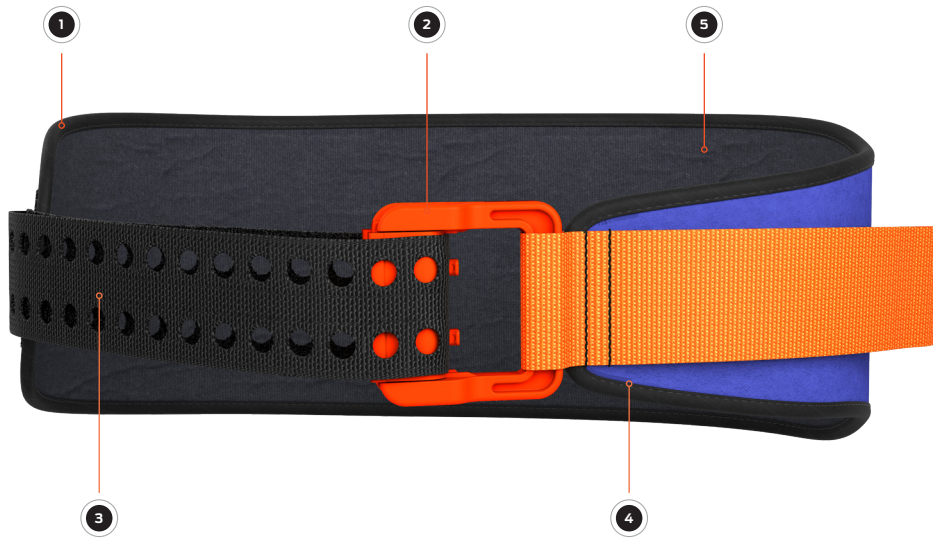

### MODELS



CIVILIAN  
ORANGE/BLUE



MILITARY  
OD GREEN /BLACK

## PRODUCT SPECIFICATIONS

### PACKAGED SIZE & WEIGHT\*

Height: 5.5 in (14 cm)  
 Width: 7.5 in (19 cm)  
 Depth: 3.75 in (9.5 cm)  
 Weight: 8.6 - 9.2 oz (244 - 250g)\*\*

### OPERATING PRINCIPLE

Optimal force is reached when 150N of circumferential pressure is attained, which extends prongs from SAM<sup>®</sup> AUTOSTOP Buckle through the Precision Strap.

### MODELS AVAILABLE

- Civilian
- Small 27-45"/ 69-114 cm (PS300-OB-EN)
  - Standard 32-50"/ 81-127 cm (PS301-OB-EN)
  - Large 36-54"/ 91-137 cm (PS302-OB-EN)

- Military
- Standard 32-50"/ 81-127 cm (PS301-OD-EN)

### GLOBAL SALES

+1.503.639.5474  
[sammedical.com](http://sammedical.com)



Ergo Europe, Westervoortsedijk 60  
 6827 AT Arnhem, The Netherlands



SAM Medical Products<sup>®</sup>  
 12200 SW Tualatin Rd., Ste 200  
 Tualatin, OR 97062 USA

\* Size and weight vary by configuration and manufacturing process.  
 \*\* Weight varies by size (Small, Standard, Large)  
 SAM<sup>®</sup> is a registered trademark in the United States and other countries.  
 ©2024 SAM Medical Products. All rights reserved.