## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

• Mild MCL and/or LCL sprains, mild arthritis, and mild instabilities. Soft good/semi rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS

Phlebitis patients should not wear neoprene products. Do not use if you are allergic to any of the materials contained in this product

### WARNINGS AND PRECAUTIONS

• If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.  
• Do not use over open wounds.  
• Do not use this device if it was damaged and/or package has been opened.  
• If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.  
**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

1a) Unfasten and loosen the straps at the thigh and calf.

1b) Wrap product around leg and fasten over the front of the leg. Position the sleeve so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the knee cap.

2) Adjust posterior (back) thigh and calf straps for snug fit. (This should be a one time adjustment)

3) Fasten the anterior (front) thigh and calf support straps.

### CARE:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Hinges may be removed for washing. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water.

 FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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